




prospera

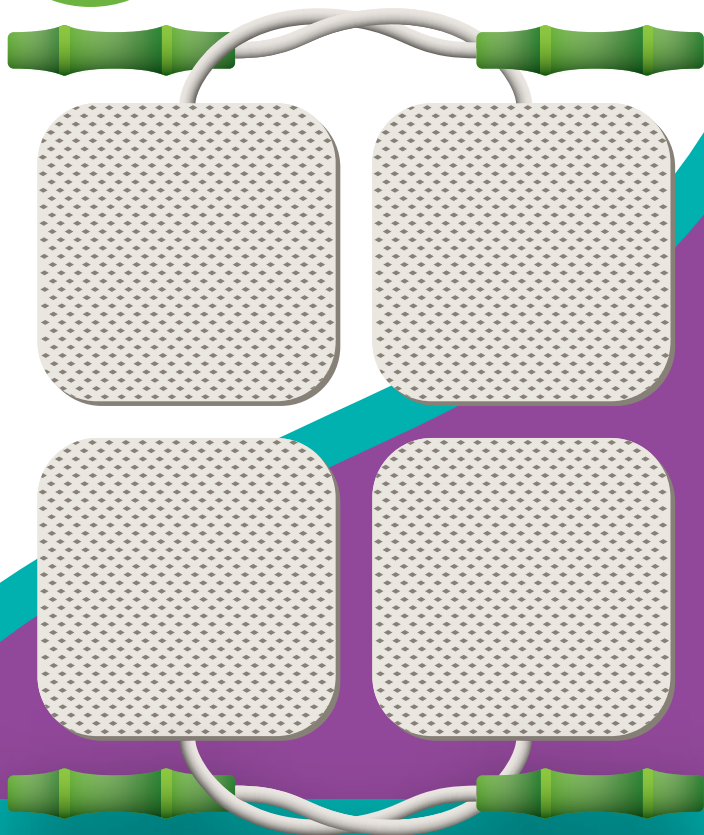

PL009-P Electronic Pulse Massager Pads

Set of 4

Application

1. Clean skin thoroughly prior to each application of electrodes. Electrodes will not stick well if any lotion, oil make-up, dirt, etc. is left on the skin.
2. Turn the massager off before you apply the electrodes to your skin. Insert the pins of the lead wire from the massager into the electrode wire connectors.
3. To remove the electrodes from the protective liner, lift at the edge of the electrodes and peel from the liner; then apply firmly to the skin.
4. Adhesion improves when electrodes reach skin temperature. Retain the protective liner in bag for storage after use.
5. To improve the life and adhesion of the electrodes, moisten the skin with a damp cloth prior to application.

Removal

1. Turn the massager off before you remove the electrodes from the skin.
2. Lift at the edge of the electrodes and peel. Do not pull on the lead wires when removing electrodes from the protective liner or skin. You may damage the electrodes if you do so.
3. Place the electrodes on the liner and remove the lead wire by gently twisting and pulling at the same time.

Care & Storage

Between uses, store the electrodes in the resealable bag in a cool, dry place.

The life of the electrodes varies depending on skin conditions, storage, climate, amount of use, type of stimulation and stimulation site. Electrode life may be extended by carefully following the application instructions.

About Prospera

Prospera develops high quality, innovative, affordable FDA approved handheld massage and pain relief devices for active people.

Prospera products are available through leading department stores, consumer electronics stores, and major online retailers.

Learn more about Prospera at www.prosperacorp.com



PL009-P Replacement Pads. Proprietary technology. Electric pulses evenly distributed. A set of four, perfect for the PL009 Electronic Pulse Massager. Compact, light weight, easy to carry and use.

